User instruction RFSU

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EN: User information

1. Handle the condom carefully. Open the package as shown in the drawing. Avoid sharp objects such as sharp nails, jewelry etc. when handling and opening. First roll up the condom a bit so that it is clearly visible in which direction it should be rolled on. Check the expiration date on the condom wrapper before using the condom.
2. Squeeze the container between your thumb and forefinger to remove air. Before intimate contact occurs between the penis and your partner, roll the condom on the erect penis to prevent pregnancy or the spread of sexually transmitted infections.
3. Stop and check that the condom is in place.
4. Immediately after ejaculation, the penis is pulled out and the condom is carefully held in place at the root of the penis so that no sperm flows out.
5. Consult a doctor if you are taking any medicine (e.g. ointment) which, if it comes into contact with the condom, may pose a risk of impaired function.
6. If the condom breaks during use, contact healthcare within 72 hours.

Safety information

The condom is the only contraceptive that protects against both pregnancy and sexually transmitted diseases. Condoms are used by sexually active people as a contraceptive and/or in the prevention of sexually transmitted infections (STIs) such as HIV and other common sexually transmitted diseases. If you wish to use extra lubricant, water- or silicone-based lubricant is recommended. Oil-based products (e.g. baby oil, vaseline and some pessaries) can damage the condom. Keep out of reach of children.

Contraindications

The condom should not be used if pregnancy is desired or if there is an allergy to latex, silicone oil or any of the flavoring substances included. The manufacturer and the competent authority must be informed of all serious incidents that have occurred with the product. No contraceptive is 100% safe.

When you buy RFSU's products, you contribute to the work with sexual education and human rights in the Nordics and large parts of the world.

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EN: Read the IFU (instruction for use) before use.

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EN: For single use only. Reuse can increase the risk of failure or infections.

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EN: Do not use the condom if the outer packaging is visibly damaged. If this is the case, you should discard it and use a new condom from an undamaged package.

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EN: This product contains latex (natural rubber), which may cause allergic reactions and, in the worst case, anaphylactic shock.

BANNER: FOR THE FREEDOM OF EVERYBODY

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We believe something better is ahead.
A world where you are free to express your
sexuality, love who you want, and decide over
your body. That's a struggle worthwhile.
Since 1933, we have been working to make it
real. And we won't stop. Not until we get there.
By choosing RFSU products you support our
ongoing work for sexual and human rights.
For a world of greater freedom, for every body.
www.rfsu.com

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EN: RFSU Grande XL are extremely spacious condoms with an extra generous shape that provides a better fit and feel.

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EN: Tight are condoms with a tight fit.

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EN: Beyond Thin are sensationally thin condoms that provide an optimal feel that is close, natural and pleasurable – almost like you are not wearing one.

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EN: Enjoy is a mix of different condoms for those who want to try and feel their way around.

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EN: A mix of the most popular flavoured and scented condoms for those who want multiple sensory stimulation: 1 pc Profil Juicy Grape, 1 pc Profil Banana split, 2 pc. Sensual Honey Melon, 2 pc Sensual Cola, 2 pc Regular Strawberry and 2 pc Regular Va

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EN: Transparent, straight and smooth condom without reservoir and lubricant.

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EN: Sensitive are extra thin, fragrance-free plastic condoms that conduct body heat and provide a natural feel. Can be used by those who are allergic to latex.